

W̄SÁNEĆ School Board's EW 801 Thermometer Guidelines


The infrared thermometer detects temperature at the touch of a button with an accuracy of +/- 0.2 °C. Temperature can be measured quickly and accurately in one second and if out of range the screen will turn red and the alarm will beep.

Use:

- Check the mode button to ensure the thermometer is set to body mode (three different modes include body, room, object)
- Aim the probe at the middle of the forehead
- Press the measurement button located at the front of the handle
- After one second the temperature will appear
- Retry if you were out of range and or a temperature does not appear
- Staff or Students with a temperature higher than 38 °C are not permitted on-site or to participate in Outdoor Education until symptoms resolve.

Warning:

This product uses a digital display as well as a three colour backlight function to indicate if a temperature is higher than normal and will sound an alarm if its higher than 38 °C.

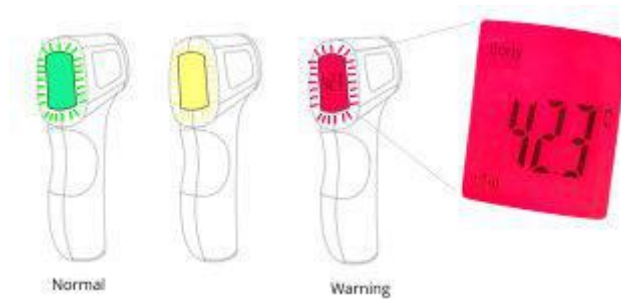

- 32.0 C - 37.4 °C - Green
- 37.5 C - 37.9 °C - Orange
- 38.0 C - 42.9 °C – Red

